


12A - 13A Seat widths 38-39-40-42-44

12B - 13B Seat widths 46-48-50

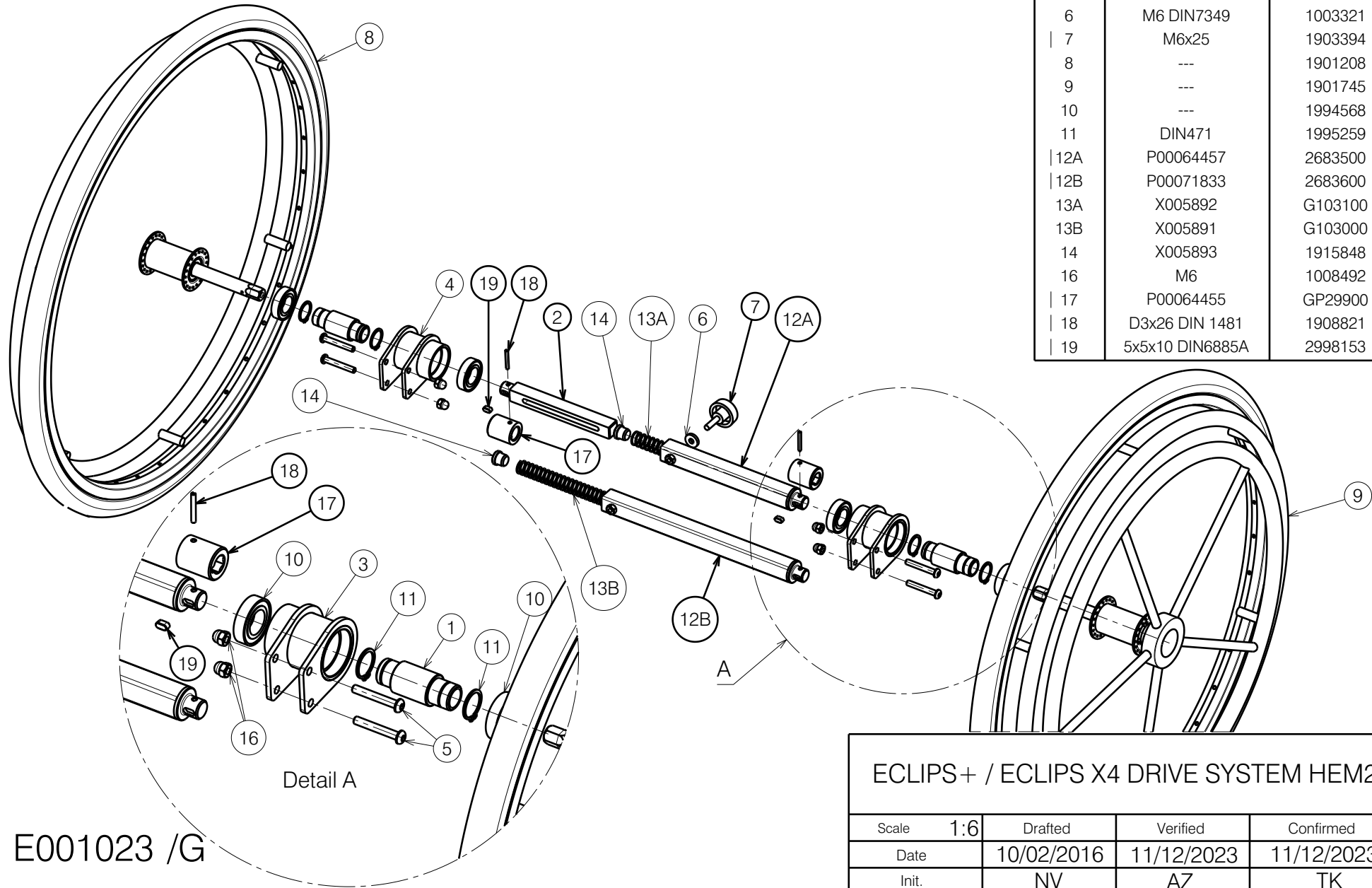

| Material |                 |         |          |
|----------|-----------------|---------|----------|
| No.      | Plan            | Article | Quantity |
| 1        | X002976         | 1995343 | 2        |
| 2        | P00065423       | 2684100 | 1        |
| 3        | Y004693         | 1518204 | 2        |
| 4        | Y004751         | 1521604 | 2        |
| 5        | M6x40 ISO7380   | 1006111 | 4        |
| 6        | M6 DIN7349      | 1003321 | 1        |
| 7        | M6x25           | 1903394 | 1        |
| 8        | ---             | 1901208 | 1        |
| 9        | ---             | 1901745 | 1        |
| 10       | ---             | 1994568 | 4        |
| 11       | DIN471          | 1995259 | 4        |
| 12A      | P00064457       | 2683500 | 1        |
| 12B      | P00071833       | 2683600 | 1        |
| 13A      | X005892         | G103100 | 1        |
| 13B      | X005891         | G103000 | 1        |
| 14       | X005893         | 1915848 | 1        |
| 16       | M6              | 1008492 | 4        |
| 17       | P00064455       | GP29900 | 2        |
| 18       | D3x26 DIN 1481  | 1908821 | 2        |
| 19       | 5x5x10 DIN6885A | 2998153 | 2        |

E001023 /G

ECLIPS+ / ECLIPS X4 DRIVE SYSTEM HEM2

Plan No.

E001023 /G

Scale 1:6

Drafted

Verified

Confirmed

Date

10/02/2016

11/12/2023

11/12/2023

Init.

NV

AZ

TK

